

ARE YOU MAXIMIZING YOUR POTENTIAL?

Feel the freedom of
improving your
sexual health

FDA cleared and CE-certified hand-held medical device to provoke penile erection in men who experience erectile dysfunction, and to provoke ejaculation in men with spinal cord injury.



... My erections are back and almost as strong as before surgery. **-G.A., Greencastle, PA**

I love the 2nd and 3rd erections back to back! I feel like I am in college again.

-Ron, 49 years old, NY


I recently got my updated Viberect back and what a difference! They have increased the amount of movement that the pads have which has made a huge difference for me. I was getting about 60 to 65% erection at best with old unit and seemed to get back some sensations I had lost. With updated version so far I have had near 100% erections every time I have used it. This is the first time without injections or VED since Oct 2007. The slower speed setting actually has worked the best for me.

-C.T., WV

BELIEVE IN YOUR BODY'S POTENTIAL!

Penile Vibratory Therapy Can Help Treat Men With ED & Ejaculatory Dysfunction

1. Improve Nerve Conduction to Penis
2. Improve Blood Flow (Pudendocavernosus Reflex and Bulbocavernosus Reflex)
3. Help Strengthen Muscles Around the Penis That Act as Physiological "Penis Rings"

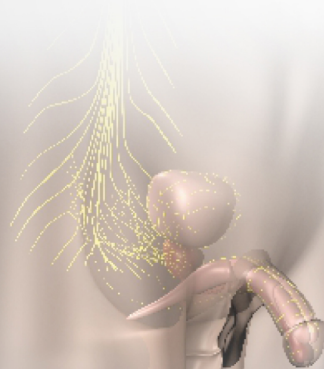

POWERFUL FREQUENCY & AMPLITUDE SETTINGS TO PROVOKE EJACULATION IN MEN WITH SPINAL CORD INJURY

EFFORTLESS OPERATION

Certifications:

FDA (USA), CE (European Union), ISO 13485:2003, CMDCAS, Health Canada, TGA (Australia), EN 60601-3

CE Indications:

Penile vibratory stimulation device for the treatment of erectile and ejaculatory disorders

FDA Indications:

Class II medical device for home and clinic use to provoke penile erection in men with erectile dysfunction, and to provoke ejaculation in men with spinal cord injury.

VIBERECT INDIA

 **(+91) 9810629280**
info@viberectindia.com

 To purchase, go to **VIBERECTINDIA.COM**
Shipping available worldwide

VIBERECTINDIA.COM

YOU NOW HAVE
**A PHYSIOLOGICAL
ALTERNATIVE**

VIBERECT[®]

MADE IN USA  **CERTIFIED**


NEW & IMPROVED
**MORE POWERFUL
THAN EVER**



POWERED BY **VIBERECT
INDIA**

HOW IT WORKS

Penile erection is a nerve reflex. Millions of nerve receptors line the surface of your penis. Their stimulation trigger centers in the spinal cord and brain to start a process that lead to penile erection, firmness and ejaculation. With specific frequency and amplitude, the Viberec® safely stimulates these sensors. No medications, pumps, or needles are necessary. Through gentle and painless vibrations, Viberec helps initiate the flow of blood into your penis and stimulates the muscles that keep it there longer, making it more rigid. If the erections are short-lasting and firmness is a problem, try Viberec®.

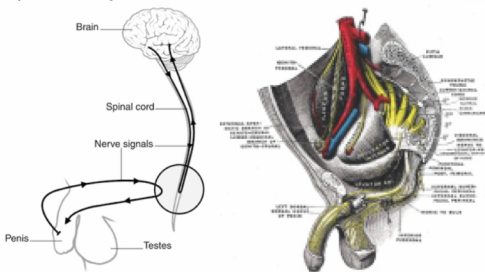

SAFE PHYSIOLOGICAL EFFECTIVE

VIBERECT-ED & VIBERECT-X3 combine Vibratory stimulation technology with body's natural functions and nerve reflexes to provoke penile erections and other sexual responses.

TOP


6 Ways to Prevent Erectile Dysfunction

- 1 Avoid Smoking
- 2 Frequent Aerobic Exercise
- 3 Avoid Triggers of Anxiety & Depression
- 4 Maintain Healthy & Meaningful Relationships
- 5 Eat Healthy
- 6 Keep Your Nerve Reflexes Tuned & Strengthen Your Erectile Muscles With Viberec

Genital Vibratory stimulation is an established modality that has been used for decades for treatment of sexual dysfunction in both men and women.

YOU DESIRE LONG-LASTING ERECTION & RIGIDITY FOR YEARS TO COME?

Erection Is A Reflex.
Stimulate Them With Viberec®

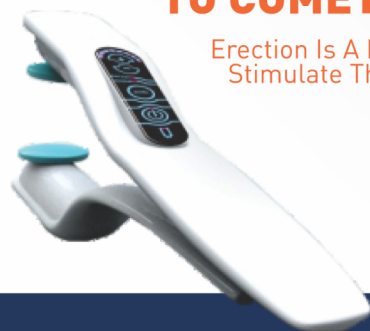


At Reflexonic, we have it down to a science.




THE ANSWER LIES IN YOU

STEP-BY-STEP REFLEXONIC® PRE-VIBERECT® STRETCH EXERCISES & PROPER APPLICATION OF THE VIBERECT® FOR OPTIMAL SEXUAL RESPONSE

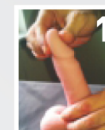


1 Lie on your back or set comfortably.

Grasp the head of penis as demonstrated.

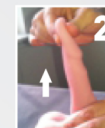


2 Squeeze the head and stretch the penis straight in the air until its natural limit.

You should feel a gentle reflex tightening of your pelvis and penis.

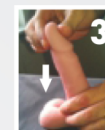


3 Release the stretch back to normal length.

Do not release your grip. Repeat steps 2 & 3 very 2-3 seconds for 2-3 minutes. The penis gradually becomes engorged with arterial blood and gets incrementally rigid, exposing nerve antennas for Viberec stimulation.

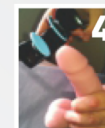


4 Hold the Viberec® close to the vibrating end and bring the medical Softpads toward the head of the penis.

Attempt different positions. Standing is a very effective position, with penis located below the heart

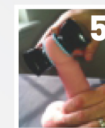

5 As you close your grip, the Viberec® automatically activates. Apply pressure.

You can move the Viberec® to find the most sensitive area.

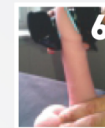

6 Now stretch your penis

While stimulating with the Viberec®, hold for several minutes (5-10 minutes).

Note: As you stimulate, focus on sexual thoughts and intense past memories, or actual sexual encounters in your life. Let both your penis and the brain stimulate sexual centers in your spinal cord. Relax. Avoid impatience and anxiety.

Warning: Use these instructions at your own risk. Never perform pre-Viberec® & Viberec® stimulation if you have sores or tears on your penis. Viberec® use in men with spinal cord injury carries risk of autonomic dysreflexia.

VIBERECTINDIA.COM